

**DIRECTIONS – ISTRUZIONI – PRÉPARATION - GEBRAUCHSANWEISUNG-
INSTRUCCIONES – INSTRUKTIONER – INSTRUÇÕES - ΟΔΗΓΙΕΣ****ChromArt CRE SUPPLEMENT****REF** 4240082

10 X 5 mL

IVD **CE****INTENDED USE**

Freeze-dried selective supplement to be used with CRE-ESBL Base (REF 4080252) for the detection of carbapenems resistant Gram-negative bacteria in clinical specimens.

VIAL CONTENTS

Antimicrobials mix.....0.26 g/vial

CONTENTS/CONTENUTO/PRESENTATION/PACKUNGSINHALT/CONTENIDO/ INNEHÅLL /CONTEÚDO/ ΠΕΡΙΕΧΟΜΕΝΑ

10 VIALS FOR 500 ML OF MEDIUM / 10 FLACONI PER 500 ML DI TERRENO / 10 FLACONS POUR 500 ML DE MILIEU / 10 RÖHRCHEN FÜR 500 ML NÄHRBODEN / 10 VIALES PARA 500 ML DE MEDIO / 10 AMPULLER, VARJE AMPULL ANVÄNDS TILL 500 ML MEDIUM/ 10 FRASCOS PARA CADA 500 ML DE MEIO / 10 ΦΙΑΛΙΔΙΑ, ΕΚΑΣΤΟ ΓΙΑ 500 ML ΜΕΣΟΥ.

STORAGE / CONSERVAZIONE / CONSERVATION / LAGERUNG / CONSERVACIÓN / LAGRING / CONSERVAÇÃO / ΑΠΟΘΗΚΕΥΣΗ

Store at 2-8° - When stored as directed the supplement remains stable until the expiry date shown on the label. Do not use beyond stated expiry date. / Conservare a 2 - 8°C- In queste condizioni il prodotto rimane valido fino alla data di scadenza indicata in etichetta. Non utilizzare dopo la data di scadenza. / Conserver à 2-8°C, selon les conditions de conservation indiquées, le produit est stable jusqu'à la date d'expiration mentionnée sur l'étiquette. Ne pas utiliser après la date d'expiration. / Lagerung: 2-8°C. Das Supplement st bei vorschriftsmäßiger Lagerung bis zum aufgedruckten Verfalldatum haltbar. Verfalldatum beachten. / Conservar entre 2-8°C. Estable bajo estas condiciones, hasta la fecha de caducidad que figura en la etiqueta. No utilizar si ha caducado. / Lagras i 2-8°C. Om lagring sker korrekt håller sig supplement hela tiden fram till utgångs datumet som står antecknat på etiketten. Använd ej produkten efter utgångs datum. / Armazenar a 2-8°C. Quando armazenado nestas condições, o suplemento permanece estável até a data de expiração indicada no rótulo. Não utilizar após a data do vencimento. / Φυλάξτε στους 2-8°C. Όταν φυλάσσεται σύμφωνα με τις οδηγίες, το συμπλήρωμα παραμένει σταθερό μέχρι την ημερομηνία λήξης που φαίνεται στην ετικέτα. Μην χρησιμοποιείτε πέραν της αναφερομένης ημερομηνίας λήξεως.


**PRECAUTIONS / PRECAUZIONI / PRÉCAUTIONS / VORSICHTSMABNAHMEN / PRECAUCIONES / SÄKERHETSFÖRESKRIFTER
PRECAUÇÕES / ΠΡΟΦΥΛΑΞΕΙΣ**

For *in vitro* diagnostic use only. The supplement should be used only by adequately trained personnel with knowledge of microbiological techniques in the laboratory. / Solo per uso diagnostico *in vitro*. Il prodotto deve essere usato solo in laboratorio, da operatori addestrati e con conoscenze delle tecniche microbiologiche di base. / Pour usage *in vitro* exclusif. Ce produit ne doit être utilisé qu'en laboratoire par des personnels formés aux techniques de microbiologie. / Zum Gebrauch für *in vitro* Diagnostik. Nur entsprechend ausgebildete Personen mit Kenntnissen in mikrobiologischen Methoden dürfen dieses Produkt nutzen. / Uso sólo para diagnóstico *in vitro*. El suplemento debería ser sólo usado por personal adecuadamente cualificado con conocimientos de las técnicas microbiológicas en laboratorios. / Endast för *in vitro* diagnostik. Supplementet skall endast hanteras av behörig personal med kunskap om mikrobiologiska laboratorietekniker. / Somente para uso diagnostico *in vitro*. O suplemento deve ser utilizado somente no laboratório por pessoas adequadamente treinadas e com conhecimento em técnicas microbiológicas. / Το συμπλήρωμα πρέπει να χρησιμοποιείται από κατάλληλα εκπαιδευμένο προσωπικό με γνώση των μικροβιολογικών τεχνικών στο εργαστήριο.

DIRECTIONS / ISTRUZIONI / PREPARATION / ZUBEREITUNG / INSTRUCCIONES / INSTRUKTIONER / INSTRUÇÕES / ΟΔΗΓΙΕΣ.


5 mL
Sterile distilled water
Acqua distillata sterile
Eau distillée stérile.
Steriles destilliertes Wasser
Água destilada estéril
Sterilt destillerat vatten
Água destilada estéril
Αποστειρωμένου αποσταγμένου νερού

Mix well
Mescolare bene
Mélanger bien
Gut mischen
Mezclar bien
Blanda väl
Misturar bem
Αναμίξτε ήπια

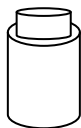

The solution is homogeneously turbid
La soluzione è omogeneamente torbida
La solution est uniformément turbide
Die Lösung ist gleichmäßig trübe
La solución es uniformemente turbia
Lösningen är likformigt grumlig
A solução é uniformemente turvo
Η λύση είναι ομοιόμορφα θολό

500 ml - 45°C / 50°C
Sterile medium / Terreno sterile /
Milieu stérile / Steriles medium /
Medio estéril / Sterilit medium /
Meio estéril / Αποστειρωμένο μέσο

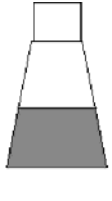

ChromArt CRE-ESBL BASE (REF 4080252)

Mix well and distribute into sterile Petri dishes.
Mescolare e distribuire in piastre sterili.
Bien mélanger et répartir
Gut mischen und Platten gießen
Mezclar bien y verter en placas Petri estériles.
Blanda väl och tappa upp i sterila Petri skålar
Misturar bem e distribuir em placas de Petri estéreis.
Ανακατέψτε καλά και περιχύστε σε αποστειρωμένα



www.biolifeitaliana.it